

Assembly

The larger wheelchairs have been sent in two pieces, top and bottom and the wheels have been removed. The smaller wheelchairs have been shipped with the wheels removed. Please follow the instructions below for assembly.

Tools needed to reassemble the top and bottom will be a phillips screw driver and a small crescent wrench.

Remove the screw and locking nut from the lower portion of the wheelchair. Replace the top and bottom sliding the smaller tube into the larger and replacing the screw and locking nut. Don't tighten the locking nut as you may have to adjust the height later. Once you have the wheelchair assembled and adjusted for the proper height be sure to tighten all screws and nuts. Periodically check for loose nuts.

In some cases the front curved arms have been removed to fit in the box. Replace these with the black webbing to the top (the one without the red neoprene).

Use the following instructions for replacing the wheels.

Once you have the wheels on the wheelchair refer to the previous page for the proper height adjustment.

To replace the wheels you will need:

☒ 1/2 wrench (2) or crescent wrench (2) or one of each.

☐ 3/4 wrench (2) or crescent wrench (2) or one of each

Working from the inside out:

- 1: place the bolt through the block
- 2: the plastic spacer or washer(s)
- 3: then the wheel
- 4: then the washer
- 5: then the locking nut
- 6: tighten locking nut until the wheel still moves freely.

